

**REF**

Product Code – Référence – Artikelnummer – Numéro de lot – Chargenbezeichnung –


Numero di lotto

**LOT**

Use by date – A utiliser avant – Verwendbar bis – Caducidad –

Data di scadenza

Sterilized by ethylene oxide – Stérilisé à l'oxyde d'éthylène – Sterilisiert mit Ethylenoxid – Esterilizado con óxido de etileno – Sterilizzato con ossido di etilene

 Do not reuse – Strict usage unique – Nur zum einmaligen Gebrauch – Válido para un solo uso – Monouso


Consult instructions for use – Lire le mode d'emploi – Lesen Sie die Gebrauchsanweisung – Leer las instrucciones de uso – Leggere le istruzioni per l'uso



Manufactured by – Fabriqué par – Hergestellt von – Fabricado por – Fabricato da



Date of manufacture – date de fabrication – Herstellungsdatum – Fecha de fabricación – Data produzione



Temperature limitation – Limite de température – Temperaturbegrenzung beachten – Limite de temperatura – Limite di temperatura



Non-pyrogenic – Apyrogène – Pyrogenfrei – Não pirogénico – Senza pirogeni



Do not use if package is damaged – Ne pas utiliser si l'emballage est endommagé – Bei beschädigter verpackung nicht verwenden – No usar en caso de envase dañado – Non utilizzare in caso di confezione danneggiata



Do not resterilize – Ne pas restériliser – Nicht erneut sterilisieren – No reesterilizar – Non risterilizzar



Caution: Federal (USA) law restricts this device to sale or on order of a physician



Caution

  
1639


# HYPERTHERMIC PERFUSION FIXED PACK

## FOR USE WITH THE THERMOCHEM™ FAMILY OF DEVICES

### Instructions for Use

#### Cautions and Warnings:

1. If using any pharmaceutical agents during the procedure please follow the hospital internal guidelines for handling and disposal of any contaminated materials or products, as well as complying with the labelled recommendations of the pharmaceutical manufacturer regarding appropriate protective clothing, handling, and disposal of any contaminated material or products.
2. The user should confirm that this pack is suitable for its intended use.
3. Leakage at any point in the circuit can cause fluid loss. Careful observation of the circuit for leaks before and during circulation is recommended.
4. Do not resterilize and do not reuse the pack or any component of the kit. It is disposable and intended for one time use only.
5. Do not modify. Product may not work as intended if altered.

#### Installation:

1. Inspect the device and sterile package carefully. Do not use if the sterile package and/or device is damaged.
2. Open the Fixed Pack and remove the contents. If providing the contents to the sterile field, use proper sterile technique to transfer contents onto the sterile field.
3. Using a single Fixed Pack cable tie, circle the tubing that you wish to secure.
4. Place the thin pointed end through the square notch on the opposite end.
5. Tighten by hand until a secure fit is obtained.
6. Confirm the Fixed Pack cable tie is secure.

IFU-1002.rev.B, 2020.12.05



**Manufactured by:**  
ThermaSolutions  
1889 Buerkle Road  
White Bear Lake, MN 55110  
USA  
+1.651.209.3900

EC	REP
----	-----

ThermaSolutions Europe BV  
Takkebijsters 41  
4817 BL Breda  
The Netherlands  
Phone: +31 76 579 11 44  
www.thermasolutions.com  
emea@thermasolutions.com