



# ThermoCasing

## Instructions for Use

### Product # ANC-3610

Contents: 3' single-use tubing wrap


Distributed by  
**ThermaSolutions** 1889 Buerkle Road, White Bear Lake, MN 55110 USA  
Phone: 1-877-952-6100  
[www.thermasolutions.com](http://www.thermasolutions.com) [info@thermasolutions.com](mailto:info@thermasolutions.com)

**To apply casing to desired tubing:**


- 1) Open ThermoCasing at slit and wrap ThermoCasing around length of tubing.



- 2) Locate the end of white release paper for the self-seal strip and peel approximately 8" of strip, revealing adhesive. Press inner edges of ThermoCasing together to seal. Repeat this step in 8" segments for entire length of ThermoCasing.

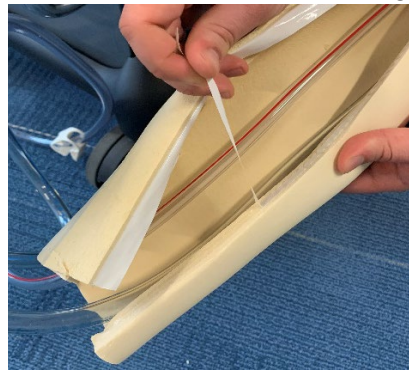

- 3) Remove the release liner on the outer flap, and apply even pressure to completely close the lap onto the outer surface of the ThermoCasing.

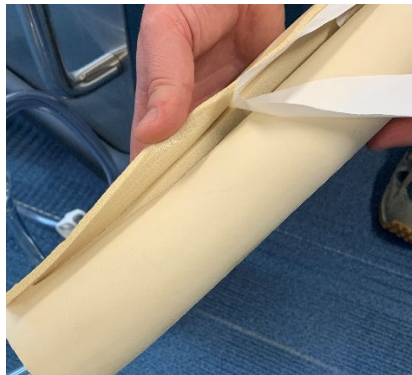

- 4) At end of case, dispose of the single-use ThermoCasing with the other disposable components.